Pigeon

**GoMini™
Plus**
ELECTRIC BREAST PUMP

Single / Double

Instruction Manual

1.0 Usage

1.1 Breast pump parts	2
1.2 Precautions for use	3-4
1.3 How to disassemble	5
1.4 How to assemble	6-8
1.5 How to use	9-10
1.6 How to store expressed milk	11
1.7 How to wash and sterilize	12-13
1.8 About the bottle and nipple	13
1.9 Troubleshooting	14
1.10 Replacement & Recycling	14
1.11 Warranty & Service	14

2.0 Product specifications	15
----------------------------------	----

E-warranty

Scan to register for warranty


EN


Meaning of symbols

	Hold to turn ON (Power On) Press once to Pause/Resume Hold to turn OFF (Power OFF)
	Caution Indicates incorrect handling that can cause injury or property damage
	Prohibited Indicates incorrect handling that can cause injury or property damage
	Protective earth (ground)
S/N	Serial number/ Lot number
	Manufacturer name
	Read the instruction manual carefully before use
	Main adapter is a class II device
	Warning Indicates incorrect handling that can cause major injury or death
	Instruction Indicates incorrect handling that can cause major injury or death
	AC
	DC
	Type BF
	Manufacturer
	No protection
	Auto shut off 30 minutes after is pressed
	For food contact

1.1 Breast pump parts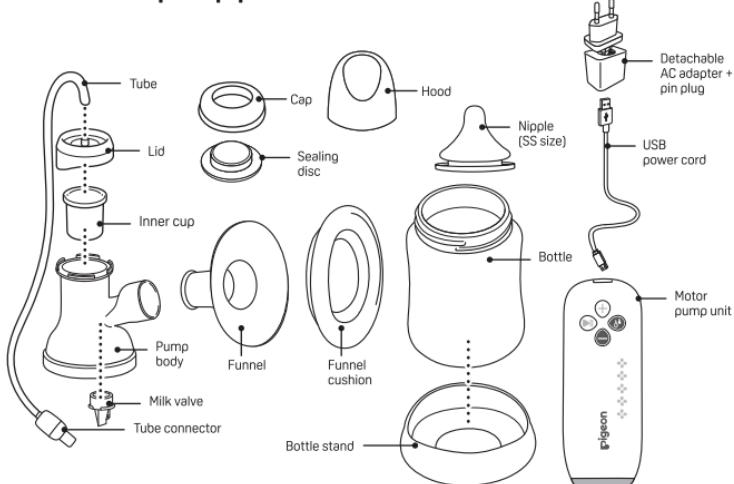

Quick start guide

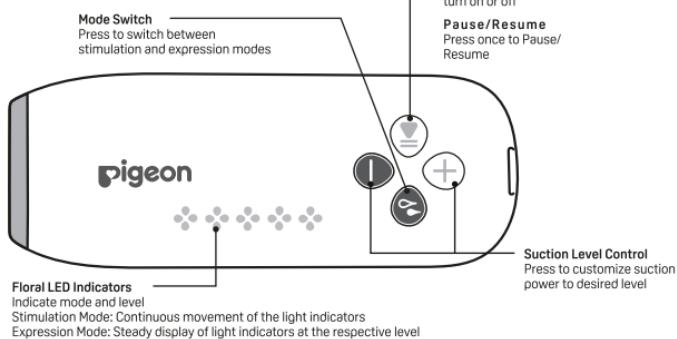Press and hold the ON/OFF button to switch on the pump. By default, it will be in the stimulation mode. When a let-down (sudden gush of breast milk) occurs, press the Mode Switch button to switch to Expression mode. The suction level can be adjusted to your level of comfort.

1.2 Precautions for use

IMPORTANT SAFEGUARDS When using electrical products, especially when children are present, basic safety precautions should always be maintained, including the following:

PLEASE READ THIS BEFORE USING THE PUMP

1. THIS APPLIANCE IS NOT INTENDED FOR USE BY PERSONS (INCLUDING CHILDREN) WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES, OR LACK OF EXPERIENCE AND KNOWLEDGE.
2. THIS APPLIANCE CAN BE USED BY PERSONS WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES OR LACK OF EXPERIENCE AND KNOWLEDGE IF THEY HAVE BEEN GIVEN SUPERVISION OR INSTRUCTION (BY A PERSON RESPONSIBLE FOR THEIR SAFETY) CONCERNING THE USE OF THE APPLIANCE IN A SAFE WAY AND UNDERSTAND THE HAZARDS INVOLVED. CHILDREN SHOULD NOT PLAY WITH THE APPLIANCE.
3. THE APPLIANCE IS ONLY TO BE USED WITH THE POWER SUPPLY UNIT (POWER PLUG & CORD) PROVIDED WITH THE APPLIANCE OR ATTACHED TO A POWER BANK WITH THE CORD PROVIDED.

The following symbols indicate incorrect handling that can result in injuries or damage.

 Warning	Indicates incorrect handling that can cause major injury or death.	 Caution	Indicates incorrect handling that can cause injury or property damage.
 Instruction	Indicates incorrect handling that can cause major injury or death.	 Prohibited	Indicates incorrect handling that can cause injury or property damage.

Danger: To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not use while bathing.
- Do not place or store this product where it can fall or be pulled into a tub or sink.
- Do not place or drop pump into water or other liquid.
- Do not reach for a product that has fallen into water. Unplug from the power socket immediately.

 Warning: To reduce the risk of burns, electrocution, fire, or injury to persons:

-  Close supervision is necessary when this product is used near children or invalids.
-  This product should never be left unattended when plugged in.
-  Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.

- 🚫 There are no user serviceable parts inside the pump. Do not attempt to open, service or repair the breast pump yourself.
- 🚫 Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- 🚫 Do not use this product if it is broken or any parts are missing.
- 🚫 Do not use while sleeping or drowsy.
- ❗ Wipe thoroughly if there is dust on the AC adapter attachment plug. Failure to do so may cause fire.
- 🚫 Do not disassemble or modify the AC adapter and motor. Doing so may cause fire as a result of electric shock or contamination.
- 🚫 Do not insert or remove the AC adapter attachment plug with wet hands. Also, do not hold motor unit with wet hands. Doing so may cause electric shock.
- 🚫 Be careful not to damage the AC adapter. (Do not scratch, modify, place near a heater, bend excessively, twist, pull, place heavy objects on, or bundle it.) Doing so may cause fire, electric shock, or product failure.
- 🚫 Never use an AC adapter other than the one provided. Doing so may cause fire, electric shock, or product failure.
- ❗ Keep the cord away from heated surfaces.
- 🚫 Do not drop or insert any object into any opening or tubing.
- 🚫 Do not allow small children or pets to play with the pump unit, AC adapter, etc.
- 🚫 Do not use outdoors with the AC adapter or operate where aerosol spray products are being used or where oxygen is being administered.


Caution

- ❗ This breast pump is intended for use by a single user only.
- 🚫 Do not use excessively or for continued suction. Doing so may cause nipple pain.
- 🚫 Stop using the breast pump if your nipples are sore. Continued use may cause your symptoms to worsen.
- 🚫 Do not wipe with thinners, benzine, or alcohol. Doing so will cause deformities and damage.
- 🚫 Do not use cleansers that contain abrasives. Doing so will cause scratches to the product.
- 🚫 Do not leave it near an open flame. Doing so will cause deformities and damage.
- 🚫 Do not subject the pump to severe shocks such as dropping or throwing. Doing so may cause malfunction or damage.
- 🚫 Do not express more than 160 ml of liquid into the bottle. The expressed breast milk may overflow.
- ❗ Prior to using the pump, check to make sure there are no abnormalities such as scratches, breaks, cracks, or deformities.
- ❗ Clean your hands before expressing.
- ❗ Wash and sterilize before using.
- ❗ The parts of the pump which come directly into contact with milk are personal care items and should not be shared between mothers.

1.3 How to disassemble

Please disassemble the parts that were pre-assembled at the time of purchase.
Do not wash or sterilize the tube and tube connector.

Pump body

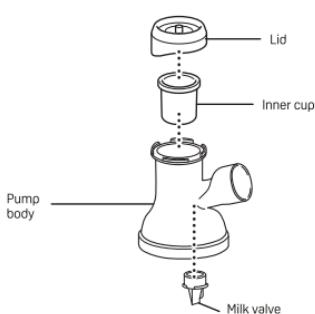


Pre-assembled at the time of purchase

1 Twist to remove the lid.

2 Remove the inner cup.

3 Remove the milk valve from the pump body.

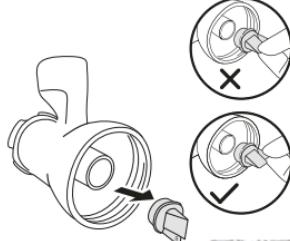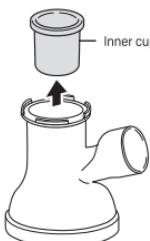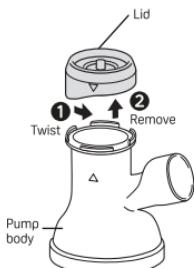Scan for
tutorial video

1.4 How to assemble

! Before assembling, dry off the parts by shaking off any excessive water after washing and sterilizing

- 1** Insert the inner cup into the pump body.

◆ Tip: Please make sure that the inner cup is not turned inside out.


Note

Please make sure that there is no moisture or dust on the inner cup before use. Moisture or dust on the inside of the inner cup may cause malfunction.

	Correct Direction	Inside Out
View from diagonally above	Thin line in the middle 	
Cross-sectional diagram		

- 2** Align the ∇ symbol on the lid with the Δ symbol on the pump body, insert it, and turn it clockwise until it stops.

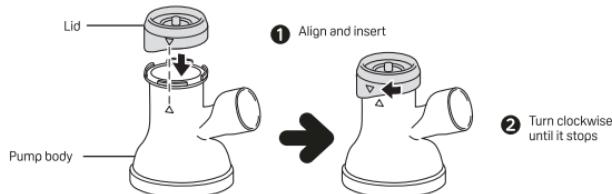- 3** Attach the milk valve to the pump body.

4 Attach the pump body to the bottle.


5 Press the bottle into the bottle stand.

◆ Tip: Using the pump with the bottle stand keeps it from toppling easily.


6 Attach funnel cushion onto the funnel.

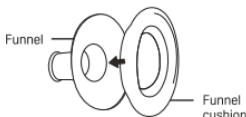7 Attach the funnel to the pump body.


8 Insert one end of the tube into the pump body.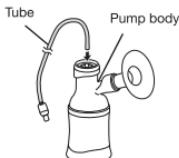

1.4 9

Insert the tube with connector into the motor.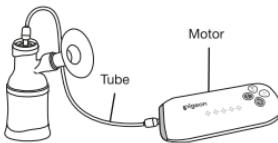

10


Plug the USB power cable to the power adapter and the micro-USB cable to the motor. Secure the pin plug to the power adapter before plugging into the socket.

◆ Tip: Other than using the power adapter to supply power, you can also connect the USB power cable to a portable power source (with output specification of 5V, 1A).

Note

The power adapter sometimes heats up during use. It is not a malfunction.

Assembly completed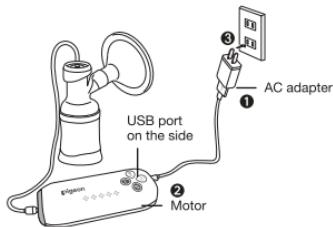




1.5 How to use

⚠ Note


This product is intended to help lactating mothers to express and collect breast milk. Contact healthcare professional or breastfeeding specialist if you have any breastfeeding related conditions/ symptoms.

Fitting the funnel properly

- Place the funnel on the breast so that the nipple is properly centered in the funnel.
- Press the funnel firmly against your breast so that air does not escape from the sides.
- You can create a good seal by using your other hand to support your breast from below.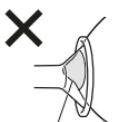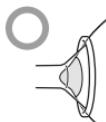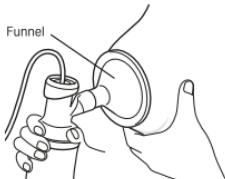


The nipple position is not centralized

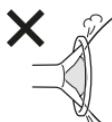
Air escapes

Expression sequence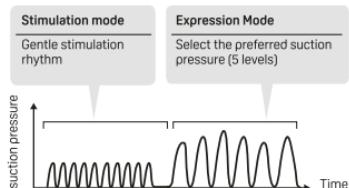

Tip

- The estimated duration for expressing each breast is 20 minutes.
- The volume of breast milk expressed varies with the condition of the breast and the individual's condition.
- Please observe your condition while expressing.

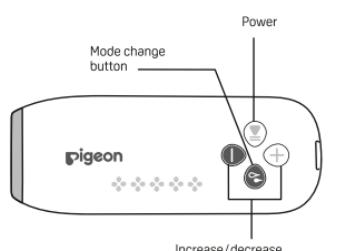⚠ Note

- Please observe your condition while you adjust the suction pressure to something that is right for you. Sudden increase in suction pressure may cause nipple pain.
- Do not over-express or express for a prolonged period. Doing so may cause nipple pain.
- It is normal to hear clicking sounds during operation. The sound emitted is due to the release of pressure from the milk valve.

Stimulation mode

Gently stimulate your breast using a slow rhythm.

When a let-down starts

Expression Mode

Select the preferred suction pressure.

Stimulation mode – 1 Level

Long press the power button to enter the Stimulation mode.

When a let-down (sudden gush of breast milk) occurs, press the button to switch to Expression mode.

Expression mode – 5 Levels

Upon selecting the Expression mode, press to select the preferred suction pressure (5 levels). The suction pressure increases for each level. Each level has 3 phases of suction power (kpa) that gradually increase and decrease.

How to operate the motor

Please wash after use.

(Refer to page 12-13)

1. Power

- Power On: Long press
- Power Off: Long press
- ※ Power cuts off automatically 30 minutes after the power is switched on.

2. Pause/Resume

- Press once to pause
- Press again to resume
- ※ Power cuts off automatically 30 minutes after the resume button is pressed.

3. Stimulation mode

- When the power is switched on, the pump automatically enters the stimulation mode.

4 . Expression mode

- Press to enter the Expression mode. (After entering this mode, the pump automatically switches to suction pressure Level 3.)
- Switching modes: Press again to return to the Stimulation mode.
- Suction pressure: Press to adjust the suction pressure.

LED Indicator

- Stimulation mode: blinks from the first floral indicator to the fifth floral indicator.
- Expression mode: lights up from the first to third floral indicator, then additional floral indicator light up or go off as you press

1.6 How to store expressed breast milk

Storage period

You are advised to feed freshly expressed breast milk to your baby as soon as possible. It is recommended not to store breast milk for longer than the duration indicated below.

Recommended Storage	Refrigerator ($\leq 4^{\circ}\text{C}$)	Freezer (about -18°C)
Freshly pumped breast milk	24 hours	3 months (recommended) 6 months (usable)
Thawed breast milk (thawed in the refrigerator or submerged in running water)	24 hours*	Do not refreeze

* If thawed by immersing in warm water, do not place back in the refrigerator.

How to thaw frozen breast milk

Thaw breast milk using method 1, 2 or 3*.

1. Immerse in warm water at around 40°C .
2. Place in the refrigerator.
 *For up to 24 hours after taken out of the freezer.
3. Place under warm running water.

*Do not refreeze thawed breast milk.

* Wipe the liquid off the outer surface of the nursing bottle with a clean cloth after thawing or warming in a lukewarm water bath.

How to warm to suitable temperature for feeding

Immerse in warm water at around 40°C . Change water when it gets cold after some time. Keep repeating process until the milk is of the right temperature for feeding.

Alternatively, you may use electric bottle warmer for convenient and hassle-free way of warming a feed

* Breast milk may separate during storage. This is normal and does not affect its nutritional content. Swirl gently before feeding.

* When going out, it is recommended that you transport the breast milk in a frozen state (without thawing it).

Note

- ➊ Do not boil or use boiling water or a microwave to thaw or heat chilled or frozen breast milk. Doing so will destroy the nutrients in breast milk.
- ➋ Do not thaw breast milk at room temperature.
- ➌ Do not mix newly-expressed breast milk with the breast milk that has been stored.
- ➍ The scale on the bottle is for reference only. If you need an accurate measurement, use a measuring cup or a measuring spoon.
- ➎ Please note that the refrigerator's temperature may vary with the frequency of opening and closing the refrigerator door, as well as position in the refrigerator.

1.7 How to wash and sterilize

Washing

Precautions

- * Please disassemble all parts before washing and sterilizing.
- * The tube and tube connector cannot be washed and sterilized. Doing so may cause a malfunction. Please wipe with a dry cloth.

Wash well with cleanser.

- Use cleanser such as the "PIGEON Liquid Cleanser".
- The breast pump contains many small parts. Please be careful not to lose the small parts during washing.
- Use a sponge brush for washing.
- For washing of the milk valves and inner cups, soak them in lukewarm water and wash using Pigeon Liquid Cleanser. Use fingers to rub gently as strong pressure might cause damage. Do not use brush when washing milk valves and inner cups (see diagram).
- Parts made of silicone rubber may be damaged or scratched if they are handled roughly. Please handle with care.
- If the motor pump unit or tube becomes dirty, please wipe with a dry clean cloth.

Sterilizing

After washing, sterilize using one of the following methods.

Boiling sterilization

- Place thoroughly washed parts in a large pot filled with water. Bring to a boil. (Sterilization time is 5 minutes after the water comes to a boil).
- If there is insufficient water, the parts may stick to the side of the pot and become deformed.
 - Do not boil excessively. Doing so can damage the product quickly.

Chemical sterilization

Use sterilization products made specifically for baby products. Follow instruction in the user's manual for these products.

1.7

How to wash and sterilize

Steam sterilization

- It is recommended to use a Pigeon steam sterilizer for steam sterilization. When using an electric steam sterilizer, please follow its operating instructions.

Other sterilization methods



- Do not use dry heat or a dishwasher/dryer to sterilize the parts. Doing so may cause deformation or damage.

* Parts are hot immediately after sterilizing. Please use caution to avoid injury.

Please assemble the parts
after washing and sterilizing.
(Refer to pages 6-8)

1.8 About the bottle and nipple

How to assemble nipple and cap

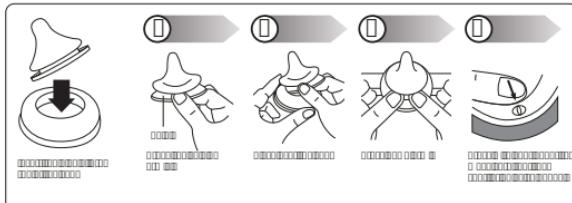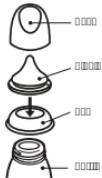1.9 Troubleshooting

Problem	Solution
Low and no suction pressure	Do ensure all parts are assembled securely. Check if the funnels are completely sealed around the breasts and also make sure your milk valves are not torn. If there is still low suction pressure, please contact Pigeon Customer Care.
Water leakage at motor unit	If you experience water leak at the bottom motor unit, you may consider to continue pumping session. After the session, disconnect tubing from motor unit and turn it on to let it run for 30 minutes to purge out remaining water in the motor unit. To prevent this problem from occurring, please do not wash the tube and ensure inner cups are not torn.
Milk leakage from the motor unit (back flow of breast milk)	If there is milk leakage from the motor unit, please turn off the motor unit and check if the inner cup is torn or damaged. If it is not torn or damaged and the motor unit is working, you may wish to continue pumping. If the motor unit is not working, please contact Pigeon Customer Care.
Motor unit is not working well or does not automatically shut off 30 minutes after switched on or after paused and resumed	Please contact Pigeon Customer Care.

1.10 Replacement & Recycling

Replacement of Pump Parts

To purchase replacement of breast pump parts, please contact Pigeon Customer Care.

Recycling

Please follow your country's rules for the separate collection of electronic products.

Correct disposal helps prevent negative consequences for the environment and human health.

1.11 Warranty & Service

1. Please scan QR code to register for warranty for this unit's electrical and electronic components. During the warranted period, Pigeon Corporation will replace any product or part thereof found to fail to meet the product specifications.
2. This warranty does not include:
 - (a) Misuse or any manner of tampering
 - (b) Alteration or modification
 - (c) Usage of wrong electrical supply or voltage
 - (d) Normal wear and tear
 - (e) Damages to any attachment not provided with the product
 - (f) Usage not in accordance with the Operation Manual
 - (g) Repair or service by any other organization except PIGEON

Scan to register for warranty

2.0 Product specifications

■ Motor

Power	5W (power adapter or external portable power source)
-------	---

Backflow Protection that will drain out
any water entering the motor unit

■ Power adapter

Input	AC100-240V 50/60Hz 0.2A Max
Output	DC5V1A

■ External portable power source specifications

Output current	1A
Output voltage	5V

Stimulation mode suction pressure/-kPa	10 ~ 13	Expression mode suction pressure/-kPa	13 ~ 34
Stimulation mode frequency (times/min)	117	Expression mode frequency (times/min)	32 ~ 85

Pigeon GoMini™ Plus Electric Breast Pump complies with IEC 60601 standard.

Under license

Pigeon Corporation

4-4, Nihonbashi Hisamatsu-cho,
Chuo-ku, Tokyo 103-8480 JAPAN
www.pigeon.com

Assembled in Indonesia

Assembled by: PT. Pigeon Indonesia
Jl. Raya Serang Km.68, Kawasan Industri Modern
Cikande Kav. 4, Serang - Banten 42186, Indonesia

Visit our website for
your guide to parenthood
and breastfeeding

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