# Wearable

## ELECTRIC BREAST PUMP SINGLE

Thank you for choosing to purchase  
Pigeon Wearable Electric Breast Pump.

Please read this instruction manual  
carefully before using the product to  
ensure proper use. Keep it in a safe  
place after reading.

### Instruction Manual

## 1.0 Usage

1.1 Breast pump parts .....	2
1.2 Precautions for use .....	3-4
1.3 Quality indication .....	5
1.4 How to disassemble .....	5
1.5 How to assemble .....	6-7
1.6 How to use .....	7-10
1.7 How to store expressed breast milk .....	10-11
1.8 How to wash and sterilize .....	12-13
1.9 Troubleshooting .....	13
1.10 Replacement & recycling .....	14
1.11 Warranty & service .....	14

## 2.0 Product specifications ..... 15

### E-warranty

Scan to register for warranty .....


EN

# Meaning of symbols


	<b>Hold to turn ON</b> (Power On) <b>Press once to Pause/ Resume</b> <b>Hold to turn OFF</b> (Power Off)
	<b>Caution</b> Indicates incorrect handling that can cause injury or property damage
	<b>Prohibited</b> Indicates incorrect handling that can cause injury or property damage
	<b>Protective earth</b> (ground)
<b>S/N</b>	<b>Serial number / Lot number</b>
	<b>Read the instruction manual carefully before use</b>

	<b>Warning</b> Indicates incorrect handling that can cause major injury or death
	<b>Instruction</b> Indicates incorrect handling that can cause major injury or death
	AC
	DC
	Type BF
<b>IPX0</b>	No protection
	Auto shut off 30 minutes after  is pressed

# 1.1 Breast pump parts

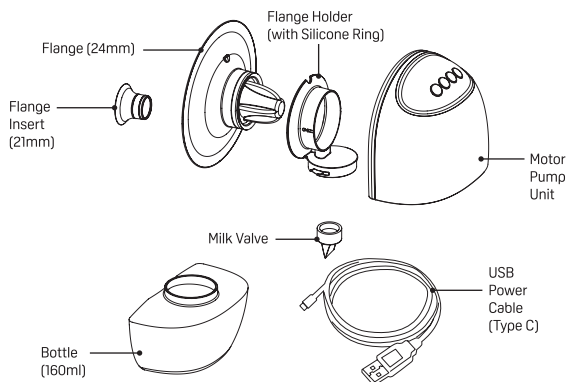## Panel Indicators
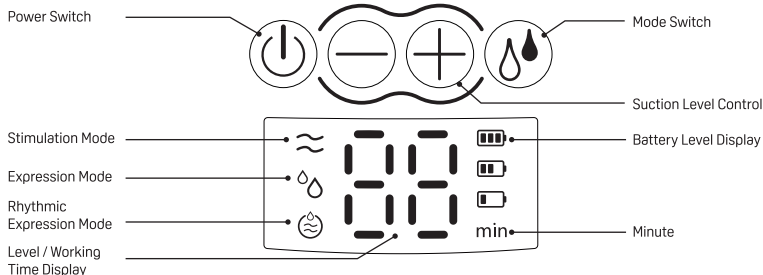
### Power Switch

Press and hold for 2 seconds to power on; press briefly to pause or restart. Press and hold for 2 seconds to power off. The device will automatically turn off after 5 minutes of inactivity or 30 minutes of operation.

### Mode Switch

Press briefly to cycle through Stimulation Mode, Expression Mode and Rhythmic Expression Mode.

### Suction Level Control

Press to customize suction strength to desired level. There are 9 levels, with L1 being the weakest and L9 being the strongest.

### Stimulation Mode

There are a total of 9 levels, with a higher frequency and lower suction strength, which can effectively stimulate the mammary glands to promote milk production.

### Expression Mode

There are a total of 9 levels, with a slower pumping frequency and stronger suction strength, suitable for rapid milk extraction.

### Rhythmic Expression Mode

There are a total of 9 levels, with rhythmic patterns of one short pulse followed by three long pulses followed by two short pulses followed by two long pulses. This stimulates the baby's sucking action to achieve a more effective pumping experience.

### Level / Working Time Display

Displays the current level setting (e.g., L5)  
Displays the current accumulated operating time (e.g., 15 min)

### Battery Level Display

Full Battery ; Battery Half Depleted ; Low Battery ;  
Flashing: Battery is about to run out

## 1.2 Precautions for use

This product is suitable for use in areas below 2000 meters above sea level.

The following symbols indicate incorrect handling that can result in injuries or damage.

 Warning	Indicates incorrect handling that can cause major injury or death.	 Caution	Indicates incorrect handling that can cause injury or property damage.
 Instruction	Indicates incorrect handling that can cause major injury or death.	 Prohibited	Indicates incorrect handling that can cause injury or property damage.

**DANGER:** To reduce the risk of electrocution:

- Keep the electric body dry, wipe with dry clean cloth when cleaning.
- Do not use while bathing or showering.
- Do not place or store the breast pump near large amount of water (e.g. sink, bathtub).
- Do not place or drop pump into water or other liquid.
- Do not reach for the product with bare hands if it falls into water; immediately unplug the power source if it is connected to a power supply.
- Do not plug or unplug the power with wet hands.
- Do not disassemble or modify the electric body.
- This is a wearable product hence the breast pump cannot be in use while charging.
- Keep the electric body and accessories away from children and pets, as these parts may pose a choking or swallowing hazard.
- Disassembling the motor unit is prohibited as any alteration or modification to it will not be covered by the warranty.
- Do not use the pump when there is visible damage on the motor unit.

 **Warning:** To reduce the risk of burns, electrocution, fire, or injury to persons:

- ⊗ Always unplug the breast pump immediately after use.
- ⊗ This product is not intended for use by individuals (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction (by a person responsible for their safety) concerning the use of the appliance in a safe way and understand the hazards involved.
- ❗ Close supervision is necessary when this product is used near children.
- ⊗ Before each use, inspect the breast pump for any visible damage. Do not use the breast pump if the plug is damaged, if it is not working properly, or if it has been dropped or immersed in water.
- ⊗ Do not allow children or pets to play with the pump unit, USB cable, etc.
- ⊗ Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- ⊗ The appliance must only be supplied at SELV.
- ⊗ If the breast pump malfunctions, do not attempt to open the motor unit for repairs as any alteration and modification to the motor unit will not be covered by the warranty. Contact Pigeon's customer service for assistance.
- ⊗ Do not attempt to remove the breast pump from the breast during operation.
- ⊗ Do not use the breast pump during pregnancy.
- ⊗ Do not use the breast pump when you are about to fall asleep or while sleeping.
- ⊗ Do not expose the breast pump to high temperature or direct sunlight.
- ⊗ Do not hold the electric pump body with wet hands.
- ⊗ If charging takes more than 6 hours and is still not complete (it takes approximately 3 hours to charge the battery from empty to full), stop charging to avoid the risk of fire, overheating, or explosion.

- ❗ Keep the USB power cable away from heated surfaces.
- ⊘ Do not drop or insert any object into any opening of the breast pump.
- ⊘ Do not operate the breast pump where aerosol spray products are being used or where oxygen is being administered.
- ⊘ Cleaning and user maintenance shall not be made by children without supervision.

### **Caution**

- ❗ This breast pump is intended for use by a single user only.
- ⊘ Do not use for purposes other than milk extraction.
- ⊘ Do not use excessively or for prolonged expression (recommended expression time is less than 30 minutes). Doing so may cause nipple pain.
- ⊘ Stop using the breast pump if your nipples are sore. Continued use may cause your symptoms to worsen.
- ⊘ Do not express more than 160ml of liquid into the bottle. The expressed breast milk may overflow.
- ⊘ Do not wipe with thinners, benzene, or high content of alcohol. Doing so will cause deformities and damage.
- ⊘ Do not use cleansers that contain abrasives. Doing so will cause scratches to the product.
- ⊘ Do not leave it near an open flame. Doing so will cause deformities and damage.
- ⊘ Do not subject the pump to severe shocks such as dropping or throwing. Doing so may cause malfunction or damage.
- ❗ Prior to using the pump, check to make sure there are no abnormalities such as scratches, breaks, cracks, or deformities.
- ❗ Clean your hands before expressing.
- ❗ Wash and sterilize before using.
- ❗ The parts of the pump which come directly into contact with milk are personal care items and should not be shared between mothers.
- ⊘ Store the product in a clean, hygienic, dry, and well-ventilated place.

## 1.3 Quality indication

This product is suitable for use in areas with an altitude of 2000 meters or below.

Parts	Material	Heat Resistance	Composition	Sterilization Method
Flange holder, Bottle	PPSU (4,4'-Monobenzene, 1,1'-Sulfonyl Group (4-Polymer) of Chlorobenzene)	180°C	100%	Boiling, Steam, Chemical, UV*
Flange, Flange Insert, Flange Holder Sealing Ring, Milk Valve	Silicone Rubber	120°C		
Motor Pump Unit	Acrylonitrile-Butadiene-Styrene Copolymer	-		DO NOT WASH OR STERILIZE
USB Power Cable	-	-		



Spare parts include: Milk valve, flange insert (if lost or damaged during use, please replace immediately)

※ The capacity of the bottle is 160ml (maximum scale capacity).


\*It is recommended to replace the product when it starts to discolour and/or turn cloudy.

## 1.4 How to disassemble


- Please disassemble the parts that were pre-assembled at the time of purchase.
- Do not wash or sterilize the motor pump unit.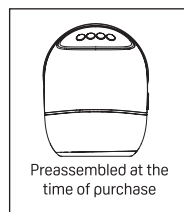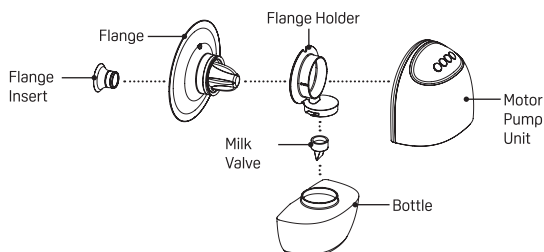


- 1** Power off the pump. Remove the breast pump and detach the motor unit. Do not invert the breast pump to prevent milk spillage.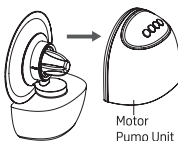


- 2** Twist to remove the bottle from the flange holder. Do not invert the bottle to prevent milk spillage.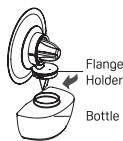

- 3** Remove the milk valve from the flange holder.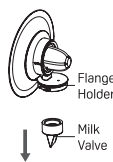


- 4** Remove the flange from the flange holder. If you are using flange insert, remove it from the flange.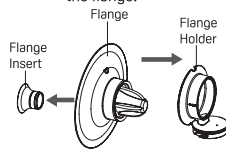


## 1.5 How to assemble

**!** Before assembling, dry off the parts by shaking off any excessive water after washing and sterilizing

- 1 Attach the milk valve to the bottom edge of the flange holder. Ensure the milk valve is fully inserted.

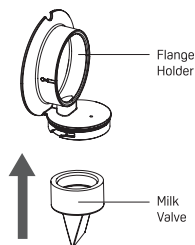
- 2 Attach the bottle to the bottom of the flange holder. Ensure the inner circle of the flange holder is facing the same direction as the curve of the bottle.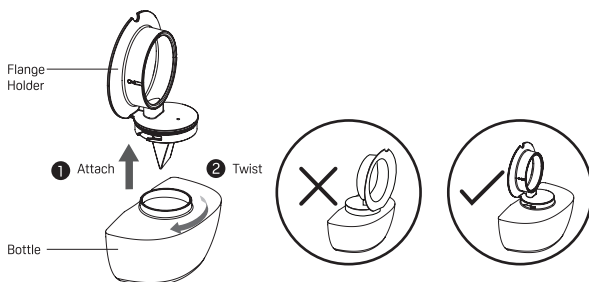


- 3 Attach the flange to the flange holder, ensuring a snug fit.


- 1 Align the circle on the flange to the hole of the flange holder.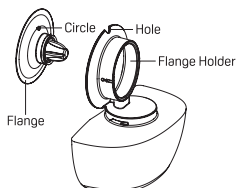


- 2 Ensure the flange and flange holder have a snug fit.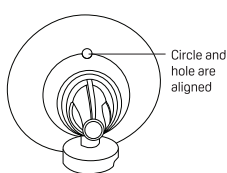


- 3 Optional – The flange insert can be used or omitted based on the size of the mother's nipple.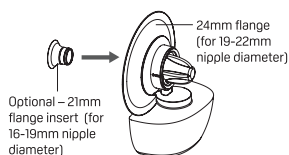

- 4 Snap the motor unit into the assembled flange.  
When you hear a “click” sound, it indicates that the two parts are securely connected.

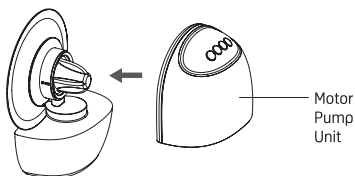### Assembly completion
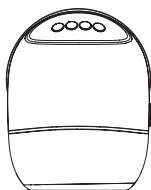
#### REMINDER

- Charging the battery to full takes approximately 3 hours (for reference). The battery can be charged repeatedly. Once fully charged, the breast pump can be used for about 2 hours (for reference).
- Before usage, ensure there are two bars of battery  for hassle-free pumping experience.
- This is a wearable product hence the breast pump cannot be in use while charging.

#### NOTE

During charging, the power adapter and motor pump unit may sometimes become warm. This is normal and not a malfunction.

## 1.6 How to use


#### Note

This product is intended to help lactating mothers to express and collect breast milk. Contact healthcare professional or breastfeeding specialist if you have any breastfeeding related conditions/ symptoms.

- Before use, wash your hands thoroughly and ensure that the breast pump components are completely clean and properly assembled according to the assembly instructions.
- This product requires a nursing bra to enable hands-free operation. Make sure to unfasten the bra straps before wearing.
- It is recommended to massage your breast before pumping.

- 1** Unclip the shoulder strap of nursing bra. Place the flange on the breast so that the nipple is properly centered in the flange. Press the flange firmly against the breast so that air does not escape from the sides. You can create a good seal by using your other hand to support your breast from below.

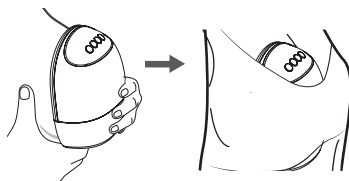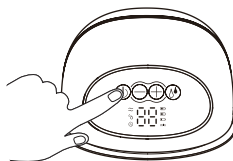
- 2** After ensuring correct placement, fasten the shoulder strap buckle of your nursing bra and adjust the strap length to a comfortable and suitable position.

Please avoid making the strap too tight, which may cause discomfort to the breasts, and also avoid making it too loose, which could prevent the breast pump from fitting properly and lead to air leaks during pumping.


◆ **Tips:**

- It is recommended to size up your bra cup size when using it with the wearable breast pump.
- Refer to following guide to find the right nursing bra size:
  - Bra should not be too tight and cause any discomfort when breast pump is placed inside the bra.
  - Breast pump does not move around in the bra.


- 3** Press and hold the power button  to turn on the pump.  
To pause during operation, press briefly the power button , and press  again to resume pumping.

- 4** On first use, the default setting is Stimulation Mode at level 4. During operation, you can press the  button to switch between three modes (Stimulation, Expression, Rhythmic Expression). On subsequent uses, the device automatically switches to the mode and level used during the last pumping session (memory mode); for example, Expression Mode at level 5.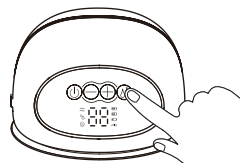

- 5 During normal operation, you can press  $\oplus$  and  $\ominus$  buttons to switch between level 1 to 9.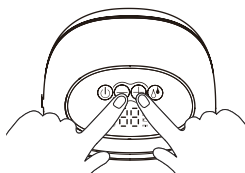

### Example




**(a)** Press and hold the  $\text{⏸}$  button to enter the Stimulation Mode. You can adjust the suction strength level by pressing the  $\oplus$  and  $\ominus$  buttons.

**(b)** Once milk begins to flow, press the  $\text{⏸}$  button to immediately switch to Expression or Rhythmic Expression Mode or wait for 2 minutes for automatic switching to Expression Mode. Adjust the suction strength according to your preference by pressing the  $\oplus$  and  $\ominus$  buttons.

**(c)** Choose between Stimulation Mode, Expression Mode or Rhythmic Expression Mode according to your needs by pressing the  $\text{⏸}$  button.

### Power connection

Open the silicone plug on the side charging port of the motor unit. Insert the Type-C end of the USB power cable into the charging port on the main unit. Plug the other end of the cable into a DC 5V/1A-3A adapter or a power outlet with a USB port (as shown in the diagram). Please note that device will not charge if charger is above 15W.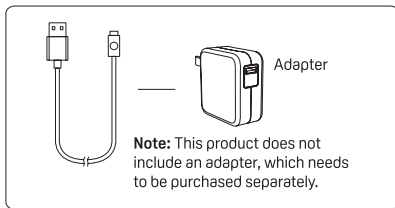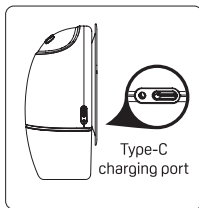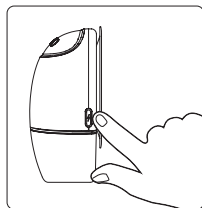

### THE DEVICE AUTOMATICALLY DETECTS THE BATTERY LEVEL.

When the battery is low, the battery indicator  $\text{⏻}$  will flash red, indicating that charging is needed. A full charge is indicated by a  $\text{⏻}$  solid light, while a half-charged battery is shown as  $\text{⏻}$ . During charging, the battery indicator will blink, and will become steady when fully charged  $\text{⏻}$ .


#### Note

- Observe and adjust the suction level to a comfortable setting while pumping. Sudden increases in suction may cause breast discomfort.
- Avoid excessive or prolonged pumping, as it may lead to breast pain.
- If you hear a “clicking” sound during use, it is normal and indicates that the milk valve is releasing suction pressure.

## Transferring expressed breast milk for feeding or storage

- 1** Power off the pump. Remove the breast pump and detach the motor unit. Do not invert the breast pump to prevent milk spillage.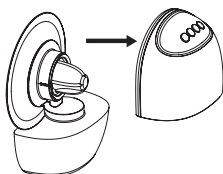

- 2** Twist to remove the bottle from the flange. Do not invert the bottle to prevent milk spillage.

- 3** Pour the milk from the bottle opening into a nursing bottle.

- 4** When pouring, you can rotate 180° to empty the remaining milk.

## 1.7 How to store expressed breast milk

### Storage period

You are advised to feed freshly expressed breast milk to your baby as soon as possible. It is recommended not to store breast milk for longer than the duration indicated below.

	Refrigerator ( $\leq 4^{\circ}\text{C}$ )	Freezer (about $-18^{\circ}\text{C}$ )
Freshly pumped breast milk	24 hours	3 months (recommended) 6 months (usable)
Thawed breast milk (thawed in the refrigerator or submerged in running water)	24 hours*	Do not refreeze

\* If thawed by immersing in warm water, do not place back in the refrigerator.


## How to thaw frozen breast milk

Thaw frozen breast milk using method 1, 2 or 3<sup>^</sup>.

1. Immerse in warm water at around 40°C.
2. Place in the refrigerator for up to 24 hours\* after taken out from the freezer.
3. Place under warm running water.

### **\* Do not refreeze thawed breast milk.**

<sup>^</sup> Wipe the liquid off the outer surface of the nursing bottle with a clean cloth after thawing or warming in a lukewarm water bath.

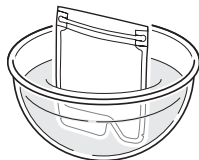## How to warm to suitable temperature for feeding

Immerse in warm water at around 40°C. Change water when it gets cold after some time. Keep repeating process until the milk is of the right temperature for feeding. Alternatively, you may use an electric bottle warmer for a convenient and hassle-free way of warming a feed.

\* Breast milk may separate during storage. This is normal and does not affect its nutritional content. Swirl gently before feeding.

\* When going out, it is recommended that you transport the breast milk in a frozen state (without thawing it).

### **⚠ Note**

- ⊗ Do not boil or use boiling water or a microwave to thaw or heat chilled or frozen breast milk. Doing so will destroy the nutrients in breast milk.
- ⊗ Do not thaw breast milk at room temperature.
- ⊗ Do not mix newly-expressed breast milk with the breast milk that has been stored.
- ⊗ The scale on the bottle is for reference only. If you need an accurate measurement, use a measuring cup or a measuring spoon.
- ⊗ Please note that the refrigerator's temperature may vary with the frequency of opening and closing the refrigerator door, as well as position in the refrigerator.

# 1.8 How to wash and sterilize


## Washing

### Precautions

- ※ Please disassemble all parts before washing and sterilizing.
- ※ The motor pump unit and the USB power cable cannot be washed or sterilized, as this may cause damage. Please wipe them with a dry clean cloth.

### Wash well with cleanser.

- Use cleanser such as the “PIGEON Liquid Cleanser”.
- The breast pump contains many small parts. Please be careful not to lose the small parts during washing.
- Use a sponge brush for washing.
- For washing of the milk valves, soak them in lukewarm water and wash using Pigeon Liquid Cleanser. Use fingers to rub gently as strong pressure might cause damage. Do not use brush when washing milk valves (see diagram).
- Parts made of silicone rubber may be damaged or scratched if they are handled roughly. Please handle with care.
- If the motor pump unit becomes dirty, please wipe with a dry clean cloth.

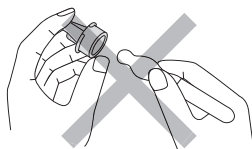## Sterilizing

After washing, sterilize using one of the following methods.

### Boiling sterilization

Bring a large pot filled with water to boil then place thoroughly washed parts into the pot to sterilize for around 5 minutes.

- If there is insufficient water, the parts may stick to the side of the pot and become deformed.
- Do not boil excessively. Doing so can damage the product quickly.

### Chemical sterilization

Use sterilization products made specifically for baby products. Follow instruction in the user's manual for these products.

### Steam sterilization

- It is recommended to use a Pigeon steam sterilizer for steam sterilization. When using an electric steam sterilizer, please follow its operating instructions.

### Other sterilization methods

- Do not use dry heat or a dishwasher / dryer to sterilize the parts. Doing so may cause deformation or damage.

\* Parts are hot immediately after sterilizing. Please use caution to avoid injury.

Please assemble the parts after washing and sterilizing.  
(Refer to pages 6-7)

## 1.9 Troubleshooting

Problem	Possible Cause	Solution
Low and no suction pressure	<ul style="list-style-type: none"> <li>• Breast pump parts are not assembled securely.</li> <li>• Flange is not completely sealed around the breasts.</li> <li>• Milk valve and/or flange are torn.</li> <li>• Battery indicator on motor unit has red light, indicating charging is needed.</li> </ul>	<ul style="list-style-type: none"> <li>• Do ensure all parts are assembled securely.</li> <li>• Check if the flange is completely sealed around the breast.</li> <li>• Ensure your milk valve and flange are not torn. If torn, replace them accordingly.</li> <li>• Do ensure your battery has a minimum 2 bar.</li> </ul> <p>If there is still low or no suction pressure after checking the above, please contact Pigeon Customer Care.</p>
Milk found on the inner side of the motor pump unit	<ul style="list-style-type: none"> <li>• Flange is not assembled correctly.</li> <li>• Flange is torn.</li> </ul>	<ul style="list-style-type: none"> <li>• Wipe the inner side of the motor pump unit dry with a clean, dry cloth.</li> <li>• Ensure the flange is assembled securely, (refer to page 6).</li> <li>• Ensure your flange is not torn. If torn, replace them accordingly.</li> </ul> <p>If motor pump unit does not work well after doing the above, please contact Pigeon Customer Care.</p>
Motor pump unit is not working well or does not automatically shut off 30 minutes after switched on or after paused and resumed	Possible malfunction of the motor pump unit.	Please contact Pigeon Customer Care.

## 1.10 Replacement & recycling

### Replacement of Pump Parts

To purchase replacement of breast pump parts, please visit Pigeon's official website in your respective country or contact Pigeon Customer Care.

### Recycling

Please follow your country's rules for the separate collection of electronic products. Correct disposal helps prevent negative consequences for the environment and human health.

#### THIS PRODUCT CONTAINS A LITHIUM-ION BATTERY

- Do not disassemble, impact, crush, or throw the motor unit into fire.
- Avoid exposing the motor pump unit to high temperatures.
- Do not attempt to replace the battery yourself; this may damage the battery, leading to overheating, fire, and personal injury.
- Do not disassemble the motor pump unit ; some components of lithium batteries are flammable and harmful, potentially causing injury, fire, rupture, or explosion.
- Do not dismantle, open, crush, or pierce the battery or battery pack with metal objects.
- Please use a certified adapter not exceeding 15W (DC 5V/1A-3A) for charging; otherwise, it may be dangerous.
- Avoid charging the motor pump unit for long hours (it takes approximately 3 hours to charge the battery from empty to full).
- Lithium batteries should only be replaced by the manufacturer or authorized service providers; using the incorrect batteries many pose explosion risks.
- To better care for and protect the environment, when you no longer need this product or its lifecycle ends, please comply with national regulations for the disposal of electronic products and hand it over to a local, nationally recognized recycling facility.

## 1.11 Warranty and Service

1. Please scan QR code to register for warranty for this unit's electrical and electronic components. During the warranted period, Pigeon Corporation will replace any product or part thereof found to fail to meet the product specifications.
2. This warranty does not include:
  - (a) Misuse or any manner of tampering
  - (b) Alteration or modification
  - (c) Usage of wrong electrical supply or voltage
  - (d) Normal wear and tear
  - (e) Damages to any attachment not provided with the product
  - (f) Usage not in accordance with the Operation Manual
  - (g) Repair or service by any other organization except PIGEON

Scan to register for warranty

## 2.0 Product Specifications

### ■ Motor Pump Unit

Power Supply	DC5V
--------------	------

#### NOTE

- Please use a power adapter with an output voltage of 5V DC and an output current of 1A to 3A.
- Some multi-stage output power adapters may not be compatible due to communication protocols, which could result in charging issues. Please replace with a suitable adapter.

### ■ Dedicated Rechargeable Battery

Battery type	Lithium Battery	Nominal Voltage	3.7V
Nominal Capacity	1400mAh	Continuous Operation Time	Approximately 2 hours (varies with usage)
Stimulation Mode Suction Strength: / -kPa	7~19	Expression Mode Suction Strength: / -kPa	10~37
Stimulation Mode Frequency: / (times/min)	67~112	Expression Mode Frequency: / (times/min)	34~100

#### Pigeon Corporation

4-4, Nihonbashi Hisamatsu-cho,  
Chuo-ku, Tokyo 103-8480 JAPAN

[www.pigeon.com](http://www.pigeon.com)

Assembled in China

Visit our website for  
your guide to parenthood  
and breastfeeding

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