# Magical Fit+

DOUBLE ELECTRIC BREAST PUMP

Please read this instruction manual prior to using the Magical Fit+ Double Electric Breast Pump to ensure correct operation. Keep it in a safe place after reading.

## Instruction Manual

### 1.0 Usage

1.1	Breast pump parts .....	2-3
1.2	Precautions for use .....	3-4
1.3	Quality indication .....	4
1.4	How to disassemble .....	5
1.5	How to assemble .....	5-7
1.6	How to use .....	8-11
1.7	How to store expressed breast milk .....	11-12
1.8	How to wash and sterilize .....	12-13
1.9	Troubleshooting .....	13-14
1.10	Replacement & recycling .....	14
1.11	Warranty & service .....	14

2.0	Product specifications .....	15
-----	------------------------------	----

EN

**15 months E-warranty**

Scan to register for warranty .....


# Meaning of symbols


	<p><b>Hold to turn ON</b> (Power On)  <b>Press once to Pause/Resume</b>  <b>Hold to turn OFF</b> (Power Off)</p>
	<p><b>Caution</b>  Indicates incorrect handling that can cause injury or property damage</p>
	<p><b>Prohibited</b>  Indicates incorrect handling that can cause injury or property damage</p>
	<p><b>Protective earth</b>  (ground)</p>
<b>S/N</b>	<p><b>Serial number / Lot number</b></p>
	<p><b>Read the instruction manual carefully before use</b></p>

	<p><b>Warning</b>  Indicates incorrect handling that can cause major injury or death</p>
	<p><b>Instruction</b>  Indicates incorrect handling that can cause major injury or death</p>
	<p>AC</p>
	<p>DC</p>
	<p>Type BF</p>
<b>IPX0</b>	<p>No protection</p>
	<p>Auto shut off 30 minutes after  is pressed</p>

# 1.1 Breast pump parts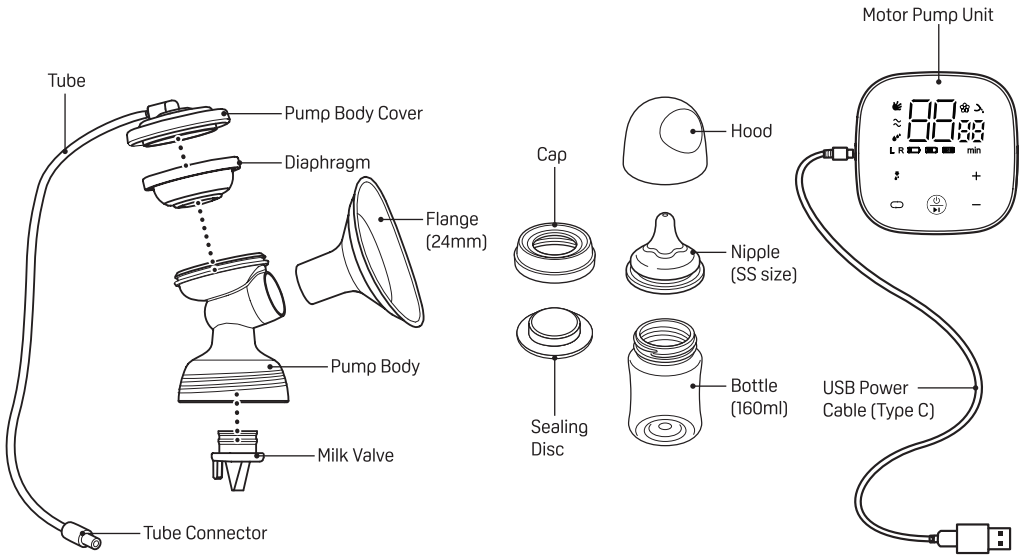

## Panel indicators

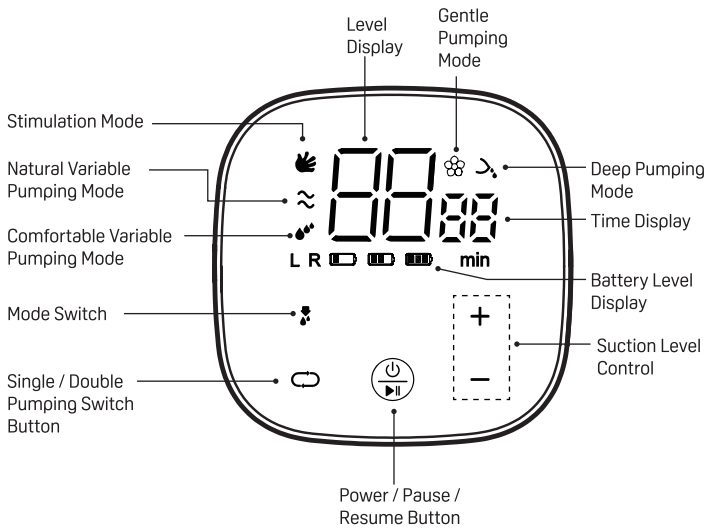**Power Switch**

Press and hold for 1.5 seconds to power on; press briefly to pause or restart. Press and hold for 1.5 seconds to power off. The device will automatically turn off after 10 minutes of inactivity or 30 minutes of operation.

**Mode Switch**

Press briefly to cycle through Stimulation Mode, Natural Variable Pumping Mode, Comfortable Variable Pumping Mode, Gentle Pumping Mode and Deep Pumping Mode.

**Suction Level Control**

Press to customize suction strength to desired level. There are 10 levels, with L1 being the weakest and L10 being the strongest.

**Stimulation Mode**

Gentle and fast suction that mimics baby's initial suckling to trigger a quick let-down.

**Natural Variable Pumping Mode**

Automatically uses a rhythm of one-short-one-long and one-short-two-long for a natural pumping experience close to a baby's natural suckling.

**Comfortable Variable Pumping Mode**

Automatically uses a rhythm of two-short-one-long and three-short-two-long for a comfortable pumping experience that supports smooth milk flow.

**Gentle Pumping Mode**

Gentle and consistent suction designed for smooth expression.

**Deep Pumping Mode**

Strong and consistent suction designed for efficient expression.

**Level Display**

Displays the current level setting (e.g., L5)

**Time Display**

Displays the current accumulated time (e.g., 05min)

**Battery Level Display**

Full Battery ; Battery Half Depleted ; Low Battery ; Flashing: Battery is about to run out 

## 1.2 Precautions for use

This product is suitable for use in areas below 2000 meters above sea level.

The following symbols indicate incorrect handling that can result in injuries or damage.

 Warning	Indicates incorrect handling that can cause major injury or death.	 Caution	Indicates incorrect handling that can cause injury or property damage.
 Instruction	Indicates incorrect handling that can cause major injury or death.	 Prohibited	Indicates incorrect handling that can cause injury or property damage.

**DANGER:** To reduce the risk of electrocution:

- Keep the motor pump unit dry, wipe with dry clean cloth when cleaning.
- Do not use while bathing or showering.
- Do not place or store the motor pump unit near large amounts of water (e.g. sink, bathtub).
- Do not place or drop motor pump unit into water or other liquid.
- Do not reach for the motor pump unit with bare hands if it falls into water; immediately unplug the power source if it is connected to a power supply.
- Do not plug or unplug the power with wet hands.
- Do not disassemble or modify the motor pump unit.
- Keep the motor pump unit and accessories away from children and pets, as these parts may pose a choking or swallowing hazard.
- Disassembling the motor pump unit is prohibited as any alteration or modification to it will not be covered by the warranty.
- Do not use the motor pump unit when there is visible damage.

**WARNING:** To reduce the risk of burns, electrocution, fire or injury to persons:

- ⊗ Always unplug the breast pump immediately after use.
- ⊗ This product is not intended for use by individuals (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction (by a person responsible for their safety) concerning the use of the appliance in a safe way and understand the hazards involved.
- ⓘ Close supervision is necessary when this product is used near children.
- ⊗ Before each use, inspect the breast pump for any visible damage. Do not use the breast pump if the plug is damaged, if it is not working properly, or if it has been dropped or immersed in water.

- ⊗ Children should be supervised to ensure that they do not play with the appliance.
- ⊗ Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- ⊗ If the breast pump malfunctions, do not attempt to open the motor pump unit for repairs as any alteration and modification to the motor pump unit will not be covered by the warranty. Contact Pigeon's customer service for assistance.
- ⊗ Please use the Pigeon standard USB power cable. Using other USB power cables may lead to fire or overheating.
- ⊗ Do not attempt to remove the breast pump from the breast during operation.
- ⊗ Do not use the breast pump during pregnancy.
- ⊗ Do not use the breast pump when you are about to fall asleep or while sleeping.
- ⊗ Do not expose the breast pump to high temperature or direct sunlight.
- ⊗ Do not hold the motor pump unit with wet hands.
- ⊗ If charging takes more than 6 hours and is still not complete (it takes approximately 3 hours to charge the battery from empty to full), stop charging to avoid the risk of fire, overheating, or explosion.
- ⓘ Keep the USB power cable away from heated surfaces.
- ⊗ Do not drop or insert any object into any opening of the breast pump.
- ⊗ Do not operate the breast pump where aerosol spray products are being used or where oxygen is being administered.
- ⊗ Cleaning and user maintenance shall not be made by children without supervision.
- ⚠ The normal temperature range for charging is 0-40°C and it must only be supplied at DC 5V/1A.
- ⚠ This product contains a Lithium-Ion Battery.
- ⚠ This product contains battery that is only replaceable by skilled persons.
- ⚠ Do not expose the product or battery to excessive temperatures.
- ⚠ Be aware of the risk of terminals of the battery-operated appliance or battery being short-circuited by metal objects.
- ⚠ Please see **1.10 Replacement & recycling** for details regarding safe disposal, disposal of used batteries, how to deal with leaking batteries.

### Caution

- ⓘ This breast pump is intended for use by a single user only.
- ⊗ Do not use for purposes other than milk extraction.
- ⊗ Do not use excessively or for prolonged expression (recommended expression time is less than 30 minutes). Doing so may cause nipple pain.
- ⊗ Stop using the breast pump if your nipples are sore. Continued use may cause your symptoms to worsen.
- ⊗ Do not express more than 160ml of breast milk into the bottle. The expressed breast milk may overflow.
- ⊗ Do not wipe with thinners, benzene, or high content of alcohol. Doing so will cause deformities and damage.
- ⊗ Do not leave the breast pump running unattended after connecting the power supply.
- ⓘ Prior to using the pump, check to make sure there are no abnormalities such as scratches, breaks, cracks, or deformities.
- ⓘ Clean your hands before expressing.
- ⓘ Wash and sterilize before using.
- ⓘ The parts of the pump which come directly into contact with milk are personal care items and should not be shared between mothers.
- ⊗ Store the product in a clean, hygienic, dry, and well-ventilated place.

## 1.3 Quality indication



Parts	Material / Composition	Heat Resistance	Sterilization Method
Flange - Soft Edge	100% TPE	110°C	Boiling, Steam, Chemical, UV*
Flange - Transparent Part, Pump Body, Cap, Hood	100% Polypropylene		
Bottle (160ml)	100% PPSU		
Milk Valves, Diaphragm, Nipple (SS Size), Sealing Disc	100% Silicone Rubber	120°C	DO NOT WASH OR STERILIZE
Tube		—	
Pump Body Cover, Tube Connector	100% Polypropylene		
Motor Pump Unit, USB Power Cable (Type C)	—		

\* The capacity of the bottle is 160ml (maximum scale capacity).


\* It is recommended to replace the product when it starts to discolour and/or turn cloudy.

# 1.4 How to disassemble


Please disassemble the parts that were pre-assembled at the time of purchase.  
Do not wash or sterilize the tube, tube connector, pump body cover and motor pump unit.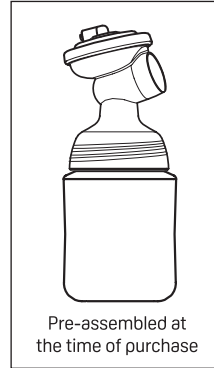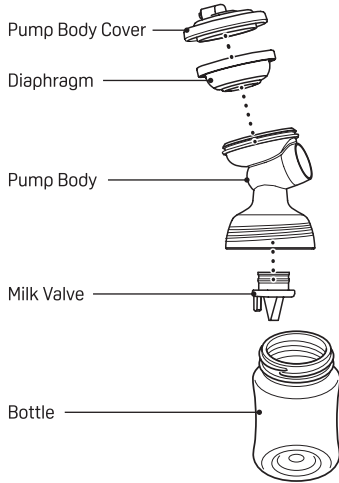


- 1** Twist to remove the bottle

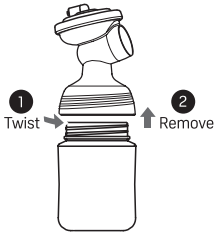
- 2** Remove the pump body cover

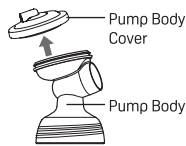- 3** Remove the diaphragm from the pump body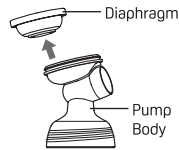

- 4** Remove the milk valve from the pump body

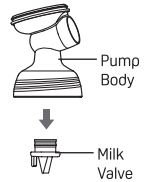
# 1.5 How to assemble


Before use, please disassemble, clean, and sterilize the breast pump parts.  
Before assembling, dry off the parts by shaking off any excessive water after washing and sterilizing.


- 1** Attach the milk valve to the pump body.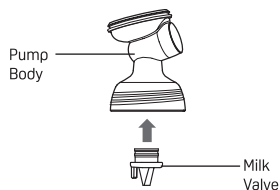

**2** Attach the pump body to the bottle.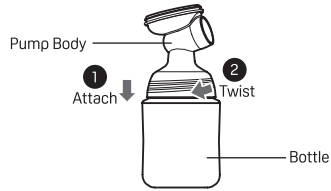


**3** Place the diaphragm into the pump body from the top and press down with your fingers to ensure it is securely fitted against all sides for an excellent seal.

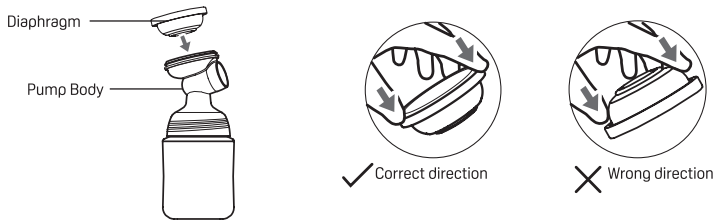
**4** Install the pump body cover over the diaphragm until it clicks into place.


**5** Attach the flange to the pump body.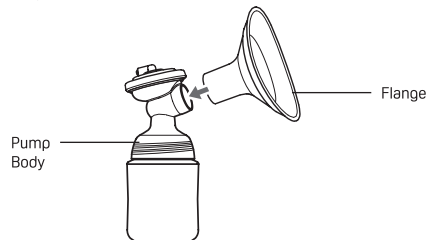


6 Insert one end of the tube into the pump body cover and the tube with connector into the motor unit.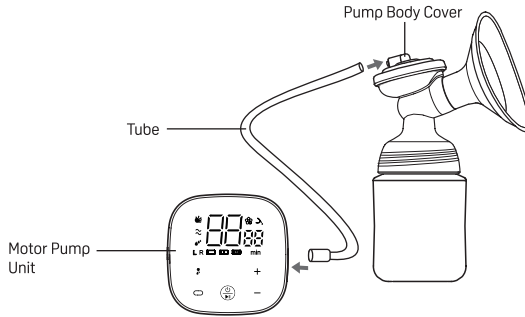

7 For single-side pumping, insert one or both tube into the air inlet of the motor pump unit. For double-side pumping, insert two tubes into the air inlets on the motor pump unit.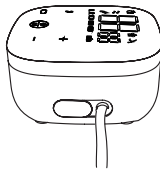


For single-side pumping

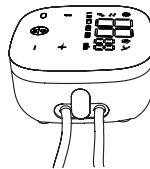For single-side or double-side pumping

## Assembly completion
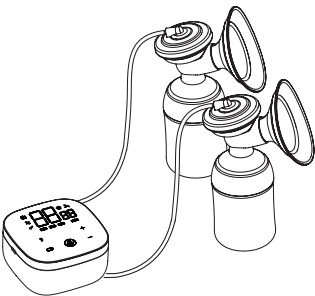
### REMINDER

- Charging the battery to full takes approximately 3 hours (for reference). The battery can be charged repeatedly. Once fully charged, the breast pump can be used for about 2 hours (for reference).
- Before usage, ensure there are two bars of battery  for hassle-free pumping experience.

# 1.6 How to use




## Fitting the flange properly

### Note


This product is intended to help lactating mothers to express and collect breast milk. Contact a healthcare professional or breastfeeding specialist if you have any breastfeeding-related conditions/ symptoms.

- Before use, wash your hands thoroughly and ensure that the breast pump components are completely clean and properly assembled according to the assembly instructions.
- It is recommended to massage your breast before pumping.

- Place the flange on the breast so that the nipple is properly centered in the flange.
- Press the flange firmly against your breast so that air does not escape from the sides.
- You can create a good seal by using your other hand to support your breast from below.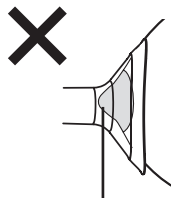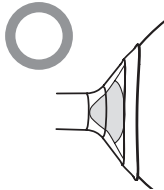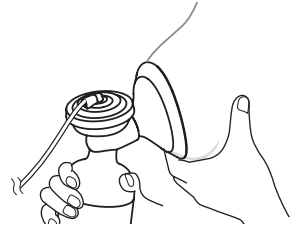


The nipple position  
is not centralized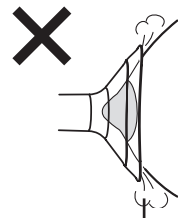

Air escapes

## Flange size

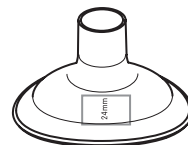- The integrated flange features a soft silicone edge and a 105° oval design that is gentle on skin, provides a better fit, and allows for more comfortable and efficient pumping, making it suitable for different breast shapes and sizes.
- The integrated flange is one-piece molded using PP (Polypropylene) and TPE (Thermoplastic Elastomer) soft rubber, which reduces the number of parts, providing great convenience in cleaning, assembling/disassembling and storage.
- This breast pump includes a default 24mm flange. If the flange is too large compared to your nipple size, please contact Pigeon Customer Care or purchase from Pigeon official website.

Default 24mm flange

## How to operate the motor unit

### 1. Power

- Power On: Long press 
- Power Off: Long press 

✳ Power cuts off automatically 30 minutes after the power is switched on

### 2. Pause / Resume

- Pause: Press  once to pause
- Resume: Press  again to resume

✳ Power cuts off automatically 10 minutes after the pause button is pressed



### 3. Switching between Double and Single Pumping

- Press  to switch between single and double pumping
- **L**: Represents pumping only on the left side
- **R**: Represents pumping only on the right side
- **L R**: Represents pumping on both left and right

### 4. Stimulation Mode

- After the power is turned on, the pump automatically enters the Stimulation Mode
- On first use, the default setting is Stimulation Mode at level 4. On subsequent uses, the pump automatically switches to the level used during the last pumping session (memory mode)
- **To adjust suction strength**: Press   to adjust between levels 1 to 10

### 5. Expression Mode

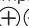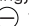- Press  to enter the Expression Mode
- On first use, the default setting is Natural Variable Pumping Mode at level 4. On subsequent uses, the pump automatically switches to the mode and level used during the last pumping session (memory mode)
- **To switch modes**: Press  to switch between the 5 modes (Stimulation, Natural Variable Pumping, Comfortable Variable Pumping, Gentle Pumping, Deep Pumping)
- **To adjust suction strength**: Press   to adjust between levels 1 to 10

### 6. LED Indicators

- **Operation Phase**: When the breast pump is running, the display screen will simultaneously show the:
  - Current pumping mode
  - Double/single pumping status
  - Suction level
  - Pumping duration
  - Battery status
- **Pause Phase**: During the pause state, the mode icon, the level, and the time display indicators on the screen will flash continuously
- **Charging Phase**:
  - While charging, one of the three lights on the battery icon on the display screen will flash
  - When charging is complete (or the battery is fully charged), all three lights on the battery icon are solid (lit up and not flashing)


(Example: Stimulation Mode, Double Pumping, Suction Level 5, Pumping Duration is 01 min and Battery Status at full bar)

	Indicates battery level 61%-100%
	Indicates battery level 31%-60%
	Indicates battery level 5%-30%
	Indicates battery level is below 5% Please charge immediately

### 7. Charging the motor pump unit

- Insert the Type-C end of the USB power cable into the charging port on the motor pump unit
- Plug the other end of the cable into a power outlet with a USB port
- The battery indicator light will turn on, and charging will begin automatically

## Expression sequence

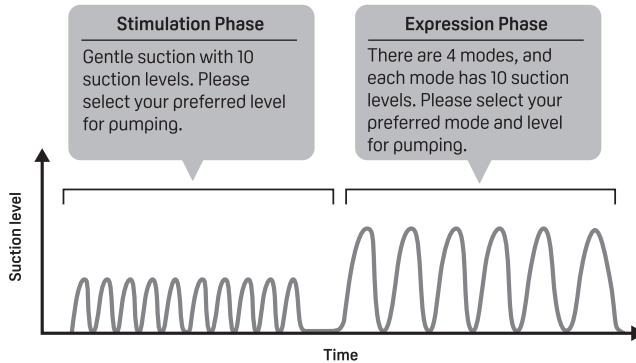### Tips

- Recommended pumping time per breast is approximately 20 minutes.
- The volume of breast milk expressed varies with the condition of the breast and the individual's condition.
- Please observe your condition while expressing.

### ⚠ Note

- Observe and adjust the suction level to a comfortable setting while pumping. Sudden increases in suction may cause breast discomfort.
- Avoid excessive or prolonged pumping, as it may lead to breast pain.
- If you hear a “clicking” sound during use, it is normal and indicates that the milk valve is releasing suction pressure.

### Example

#### Stimulation Phase

Stimulation Mode to gently stimulate your breast using gentle and fast suction

When a let-down starts  
(milk begins to flow)

#### Expression Phase

Select your preferred Expression Mode and suction level for pumping

##### Stimulation Mode – 10 Suction Levels

- Long press the power button to enter the Stimulation Mode
- Press the Suction Level Control  $\oplus\ominus$  button to adjust the suction intensity across 10 levels\*.
- Once milk begins to flow, press the Mode Switch  $\text{Ⓢ}$  button to switch to Expression Mode or wait for 2 minutes for automatic switching to Expression Mode.

##### Expression Mode – 4 Modes, 10 Suction Levels

- After entering the Expression Mode, press the Mode Switch  $\text{Ⓢ}$  button to select between different modes: Natural Variable Pumping, Comfortable Variable Pumping, Gentle Pumping, Deep Pumping
- Press the Suction Level Control  $\oplus\ominus$  button to adjust the suction intensity across 10 levels\*.

\*L1 is the lowest suction strength and L10 is the highest suction strength (As the level number increases, the suction strength increases, and the suction speed decreases)

## Overview of the 4 Expression Modes

Among the 4 Expression Modes, 2 are using variable rhythm and 2 are using fixed suction. The unique variable rhythm makes the entire pumping process easier and more comfortable while the traditional fixed suction is also retained, allowing mothers to choose a mode that best suits them.

- **Natural Variable Pumping Mode:** Uses a rhythm of one short—one long and one short—two long for a natural pumping experience close to a baby's natural suckling.
  - **Comfortable Variable Pumping Mode:** Uses a rhythm of two short—one long and three short—two long for a comfortable pumping experience that supports smooth milk flow.
  - **Gentle Pumping Mode:** Uses gentle and consistent suction for smooth expression.
  - **Deep Pumping Mode:** Uses strong and consistent suction for efficient expression.
- It is recommended to start pumping from Level 1 and gradually adjust to an appropriate suction level as your body adapts. The factory default setting is the Natural Variable Pumping Mode, and the default level for every mode is Level 4.

Please wash after use. (Refer to pages 12-13)

# 1.7 How to store expressed breast milk

## Storage period

You are advised to feed freshly expressed breast milk to your baby as soon as possible. It is recommended not to store breast milk for longer than the duration indicated below.

	Refrigerator ( $\leq 4^{\circ}\text{C}$ )	Freezer (about $-18^{\circ}\text{C}$ )
Freshly pumped breast milk	24 hours	3 months (recommended) 6 months (acceptable)
Thawed breast milk (thawed in the refrigerator or submerged in water)	24 hours*	Do not refreeze

\* If thawed by immersing in warm water, do not place back in the refrigerator.


## How to thaw frozen breast milk

Thaw breast milk using one of the following methods<sup>^</sup>:

1. Immerse in warm water at around  $40^{\circ}\text{C}$ .
2. Place in the refrigerator for up to 24 hours\* after taken out from the freezer.
3. Place under warm running water.

<sup>^</sup>Wipe off any moisture from the outside of the bottle after thawing or warming.

**\* Do not refreeze thawed breast milk.**

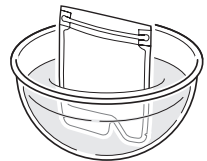## How to warm to suitable temperature for feeding

Immerse in warm water at around  $40^{\circ}\text{C}$ . Change water when it gets cold after some time. Keep repeating the process until the milk is of the right temperature for feeding. Alternatively, you may use an electric bottle warmer for a convenient and hassle-free way of warming breast milk.

### Note

- Breast milk may separate during storage. This is normal and does not affect its nutritional content. Swirl gently before feeding.
- When going out, it is recommended that you transport the breast milk in a frozen state (without thawing it).
- Do not use boiling water or a microwave to thaw or heat chilled or frozen breast milk, as this may damage nutrients.
- Do not thaw breast milk at room temperature.
- Do not mix freshly pumped breast milk with the breast milk that has been stored.
- The scale on the bottle is for reference only. If you need an accurate measurement, use a measuring cup or a measuring spoon.
- Please note that the refrigerator's temperature may vary with the frequency of opening and closing the refrigerator door, as well as position in the refrigerator. For the most stable temperature, it is recommended to store breast milk in the back of the refrigerator.

## 1.8 How to wash and sterilize


### Washing

#### Precautions

- Please disassemble all parts before washing and sterilizing.
- The tube, tube connector, pump body cover and motor pump unit cannot be washed or sterilized, as this may cause damage. Please wipe them with a dry clean cloth.

#### Wash well with cleanser.

- Use cleanser such as the Pigeon Liquid Cleanser.
- The breast pump contains many small parts. Please be careful not to lose the small parts during washing.
- Use a sponge brush for washing.
- To clean the milk valves, soak them in lukewarm water and wash using Pigeon Liquid Cleanser. Use fingers to rub gently as strong pressure might cause damage. Do not use brush when washing milk valves (see diagram).
- Parts made of silicone rubber may be damaged or scratched if they are handled roughly. Please handle with care.
- If the motor pump unit becomes dirty, please wipe with a dry clean cloth.

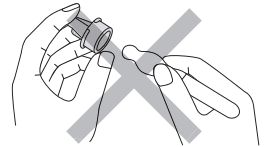### Sterilizing

After washing, sterilize using one of the following methods.

#### Boiling sterilization

Bring a large pot filled with water to boil then place thoroughly washed parts into the pot to sterilize for around 5 minutes.

- If there is insufficient water, the parts may stick to the side of the pot and become deformed.
- Do not boil excessively. Doing so can damage the product quickly.

#### Chemical sterilization

Use sterilization products made specifically for baby products. Follow instruction in the user's manual for these products.

**Steam sterilization**

- It is recommended to use a Pigeon steam sterilizer for steam sterilization. When using an electric steam sterilizer, please follow its operating instructions.

**Other sterilization methods**

- Do not use dry heat or a dishwasher / dryer to sterilize the parts. Doing so may cause deformation or damage.

\* Parts are hot immediately after sterilizing. Please use caution to avoid injury.

Please assemble the parts after washing and sterilizing. (Refer to pages 5-7)

# 1.9 Troubleshooting

Problem	Possible Cause	Solution
Low or no suction pressure.	<ul style="list-style-type: none"> <li>• Breast pump parts are not assembled securely and properly.</li> <li>• Flange is not completely sealed around the breasts.</li> <li>• Milk valves are torn.</li> <li>• Diaphragm is torn.</li> <li>• Battery indicator on motor pump unit is blinking, indicating charging is needed.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure all parts are assembled securely and properly.</li> <li>• Check if the flanges are completely sealed around the breasts (see page 8).</li> <li>• Ensure milk valves and diaphragm are not torn. If torn, replace them accordingly.</li> <li>• Ensure battery has a minimum 2 bar.</li> </ul> <p>If there is still low or no suction pressure after checking the above, please contact Pigeon Customer Care.</p>
Motor pump unit is not working well or does not automatically shut off after 30 minutes.	Possible malfunction of the motor pump unit.	Please contact Pigeon Customer Care.
The LED buttons are not working well.	Possible malfunction of the buttons.	Ensure the motor pump unit is fully charged, switch off and on the motor pump unit and try again. If it is still not working, contact Pigeon Customer Care.
Battery is not charging even after the power cable is plugged in.	Possible malfunction of the USB Type C cable or battery.	Please contact Pigeon Customer Care.

Problem	Possible Cause	Solution
The memory function of the breast pump is not working properly as it does not remember the last mode and/or suction level that was used.	Possible software malfunction.	Please contact Pigeon Customer Care.

## 1.10 Replacement & recycling

### Replacement of Pump Parts

To purchase replacement of breast pump parts, please visit Pigeon's official website in your respective country or contact Pigeon Customer Care.

### Recycling

Please follow your country's rules for the separate collection of electronic products. Correct disposal helps prevent negative consequences for the environment and human health.

#### THIS PRODUCT CONTAINS A LITHIUM-ION BATTERY

- Do not disassemble, impact, crush, or throw the motor unit into fire.
- Avoid exposing the motor pump unit to high temperatures.
- Do not attempt to replace the battery yourself; this may damage the battery, leading to overheating, fire, and personal injury.
- Do not disassemble the motor pump unit; some components of lithium batteries are flammable and harmful, potentially causing injury, fire, rupture, or explosion.
- Do not dismantle, open, crush, or pierce the battery or battery pack with metal objects.
- Avoid charging the motor pump unit for long hours (it takes approximately 3 hours to charge the battery from empty to full).
- Lithium batteries should only be replaced by the manufacturer or authorized service providers; using the incorrect batteries may pose explosion risks.
- To better care for and protect the environment, when you no longer need this product or its lifecycle ends, please comply with national regulations for the disposal of electronic products and hand it over to a local, nationally recognized recycling facility.

## 1.11 Warranty and service

1. Please scan QR code to register for warranty for this unit's electrical and electronic components. During the warranted period, Pigeon Corporation will replace any product or part thereof found to fail to meet the product specifications.



2. This warranty does not include:

- (a) Misuse or any manner of tampering
- (b) Alteration or modification
- (c) Usage of wrong electrical supply or voltage
- (d) Normal wear and tear
- (e) Damages to any attachment not provided with the product
- (f) Usage not in accordance with the Operation Manual
- (g) Repair or service by any other organization except PIGEON

Scan to register  
for warranty

## 2.0 Product specifications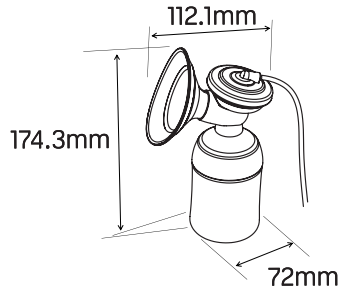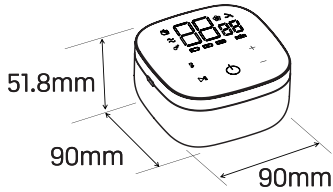

### ■ Motor Pump Unit

Power Supply	DC5V
--------------	------

### ■ Dedicated Rechargeable Battery

<b>Battery type</b>	Lithium Battery	<b>Nominal Voltage</b>	3.7V
<b>Nominal Capacity</b>	2200mAh	<b>Continuous Operation Time</b>	Approximately 2 hours (varies with usage)
<b>Stimulation Mode Suction Strength: / -kPa</b>	5~17	<b>Expression Mode Suction Strength: / -kPa</b>	8~37
<b>Stimulation Mode Frequency: / (times/min)</b>	103~130	<b>Expression Mode Frequency: / (times/min)</b>	28~181

### Pigeon Corporation

4-4, Nihonbashi Hisamatsu-cho,  
Chuo-ku, Tokyo 103-8480 JAPAN

[www.pigeon.com](http://www.pigeon.com)

Assembled in China

Visit our website for  
your guide to parenthood  
and breastfeeding

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